# RUBY 10 Safety and Care

© 2023 Freedom Scientific, Inc. All Rights Reserved. RUBY® is a registered trademark of Freedom Scientific, Inc. in the United States and other countries.

Read these instructions carefully before setting up your RUBY 10. Keep this document for future reference.

## Cleaning the LCD Screen

Before cleaning the LCD screen, power off and unplug the device. Use a dry, clean microfiber cloth to gently wipe the cloth across the screen. Optionally, use an appropriate LCD cleaning liquid and follow its instructions.

**Warning:** Do not use a paper towel. Do not press down hard on the screen.

## Cleaning the RUBY 10 Body

Unplug the device before cleaning the body. Use a lightly dampened, soft, clean cloth with water or mild detergent.

## Product Specifications

Trade name	RUBY 10
Magnification range	1.8x to 22x
Full-page magnification	0.4x to 13x
Color modes	Full color, high contrast positive, high contrast negative, yellow on black, and yellow on blue
Focus	Auto-focus
Screen	10-in TFT, 450 cd/m <sup>2</sup>
Screen resolution	1280 x 800
Camera	60 fps
Depth of focus	Approximately 20 cm
Dimensions	261 x 189 x 36 mm / 10.28 x 7.44 x 1.42 in
Weight	915 grams / 2.02 lb
Battery	Approximately 3.5 hours continuous use and 3 hours charging time
Li-ion battery	Rechargeable, 3.7 V, 7600 mAh, 28.12 Wh
Connections	USB-C, 3.5 mm audio jack
Medical power supply	Adapter Technology Co., LTD. ATM012T-W052VU
Consumer power supply	Shenzhen Fujia Appliance Co., Ltd. FJ-SW1260502500UN
Adapter input power rating	100-240 VAC, 50-60 Hz, 0.19-0.32 A
Adapter output power rating	5.1 VDC, 2.4 A

## **Warnings and Precautions**

Do not remove any parts or disassemble your unit as it will void the warranty. Contact an authorized service organization in your area.

Note these warnings and precautions:

- To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth.
- Critical diagnostic, medical or legal decisions should not be based on images displayed by this device as the device is not intended for processing of important medical, legal, or other information of a sensitive nature.
- Only use the cables provided with the unit and do not force the power connector into the device's power port.
- Do not spray any cleaning solutions directly on the LCD screen.
- Do not use alcohol, solvents, abrasives, aerosols, or any cleanser that contains acetone, ethyl alcohol, ethyl acid, ammonia, or methyl chloride.
- Do not expose the RUBY 10 to excessive heat, direct sunlight, extreme or rapid changes in temperature or humidity.
- To avoid risk of electrical damage, keep your device away from fluids and chemicals.
- Handle the RUBY 10 with care. Rough handling will damage the internal components.
- Do not use the RUBY 10 near inadequately shielded medical devices.
- Do not touch or look directly at the illumination lights.
- Be careful that your fingers and clothing do not get stuck in moving parts (swing-out arm, stand).
- To prevent possible hearing damage, do not listen at high volume levels for long periods.
- Common side effects of using an electronic display device for prolonged periods of time can lead to eye fatigue and headaches.
- Do not place the device on an unstable surface. It may fall, causing serious damage to the unit, or injury to the user.

## **Precautions for Battery Pack**

Do not charge your battery if your device appears damaged, deformed, or defective. Contact your Freedom Scientific distributor or the nearest Freedom Scientific office for further instructions prior to use.

**BATTERY MUST BE REPLACED BY CERTIFIED SERVICE PERSONNEL.**

**RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.**

**DISPOSE OF USED BATTERIES ACCORDING TO THE REGULATIONS.**

## Contacting Product Support

For product support, contact your Freedom Scientific dealer or email [support@FreedomScientific.com](mailto:support@FreedomScientific.com) or call +1 727-803-8600 weekdays between 8:30 AM and 7 PM ET. If any serious incident occurs in relation to the RUBY 10, report to Freedom Scientific and the competent authority of the country or Member State where you are established.

 Freedom Scientific, 17757 US Hwy 19 N #200, Clearwater, FL 33764, United States, [www.FreedomScientific.com](http://www.FreedomScientific.com)

 Optelec Nederland BV, Pesetastraat 5a, 2991 XT, Barendrecht, The Netherlands, [www.in.Optelec.com](http://www.in.Optelec.com)

Operating Conditions	Storage and Transportation Conditions
Temperature +10°C to 35°C / 50°F to 95°F Relative humidity < 70%, no condensation Altitudes up to 3,000 m (9,842 ft) Pressure 700 – 1060 hPa	Temperature +10°C to 40°C / 50°F to 104°F Relative humidity < 95%, no condensation Altitudes up to 12,192 m (40,000 ft) Pressure 186 – 1060 hPa

RUBY 10 is tested by a recognized body and independent laboratory for safety and regulatory compliance.

Description	Requirement
FCC / EMC	47 CFR Part 15 FCC Class B
Safety	UL/EN/CSA 62638-1:2014; IEC 60601-1:2012 reprint
CE / EMC	EN 55022:2010; IEC 60601-1-2:2014; EN 60601-1-2:2015
Usability	IEC 60601-1-6:2013, IEC 62366:2015
Risk management	ISO 14971:2019
ICC	CAN/CSA-CISPR22-10
Environmental	Consumer Product Safety Improvement Act of 2008
Environmental	FDA/OSHA/EPA regulations for toxic substances
Environmental	California Proposition 65
Safety (battery)	IEC62133-2:2017 CB report
Transportation (battery)	UN38.3 6th revision, amend 1/CNAS/SDS

## **Clinical Benefit**

Magnifiers empower the user, helping to instill independence, increase productivity, and improve self-esteem. In short, those who struggle with daily tasks due to low vision find their quality of life enhanced by utilizing magnifiers.

## **Intended Purpose**

This device is intended to be used as a vision enhancement product by people who require vision assistance when performing reading of printed materials or viewing objects encountered during everyday use. The device is not intended to be relied upon exclusively for the interpretation of sensitive or professional material, such as medical or legal documents.

## **Intended Use Environment**

The RUBY 10 is for use in a home, work, or school environment for the purpose of enlarging and enhancing images.

## **Intended User**

The RUBY 10 is intended to be used by anyone with a visual impairment.

## **Performance Characteristics**

Magnification and enhancement of images.

## **Expected Service Life**

The RUBY 10 does not create any unacceptable hazardous conditions (as defined by IEC 60601-1 Medical electrical equipment—General Requirements for Basic Safety and Essential Performance) and will be safe to use for 10 years or longer. Expected service life does not represent the actual operating life or warranty period of the device.

## **Reasonably Foreseeable Misuse**

Dropping the device from desk height. Operating the device forcefully. Using wrong accessories. Using device to read medical or legal documents.

## **Layperson Information**

For use by laypersons. Freedom Scientific is not responsible for any uses of this device other than those described in this manual.

## **Guideline**

Input rating for RUBY 10: 5 VDC, 3 A  
Protection against electric shock: Class I

## **WEEE Notice**

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on February 13, 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is the prevention of WEEE, and in addition, to promote the reuse, recycling, and other forms of recovery of wastes to reduce disposal. The WEEE logo on the product or its box indicates that this product must not be disposed of or dumped with your other household waste.

You are liable to dispose of your electronic or electrical waste equipment placing it in the specified collection point for recycling of hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure the safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, contact your local city center, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

### **Warranty Conditions**

Freedom Scientific warrants the RUBY 10, effective from the date of delivery, to be free of any defects in material and workmanship. The warranty is not transferable and does not apply to groups or multi-users. The RUBY 10 was designed for the individual purchaser to be used at home, school, and work. Freedom Scientific reserves the right to repair or replace any purchased RUBY 10 with a similar or improved product.

In no event shall Freedom Scientific or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of RUBY 10 modules. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after warranty period, contact your Freedom Scientific distributor. Use of the RUBY 10 other than described in this manual will exclude it from warranty conditions. Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. Contact your local dealer for details about the duration of this warranty.

### **Declaration of Conformity**

We declare under our sole responsibility that this product, to which this declaration relates, is in conformity with the council EMC directive 2014/30/EU and the RED directive 2014/53/EU. This device bears the CE and FCC approval mark.

### **Compliance Notice**

EU MDR 2017/745  
UL 1642, 5<sup>th</sup> Edition

This product complies with Directive 2011/65/EU of the European Parliament and of the Council of 3 January 2013, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS-II) and its amendments.

## **Susceptibility to Interference**

There may be temporary degradation of the image when the RUBY 10 HD is subjected to a strong radio frequency field, electrostatic discharge, or transient electrical noise. An electrostatic discharge (caused by static electricity) may cause the screen to go blank. If this happens, follow these steps:

1. Disconnect the DC adapter power supply from the electrical outlet.
2. Turn off the device.
3. Wait 30 seconds.
4. Connect the DC adapter power supply again and turn on the device.

## **External Influences**

Electronic devices can be sensitive to electric discharge and electromagnetic fields caused by other electronic equipment. The device should be operated at a minimum distance of one meter away from other electronic or telecommunication devices. All external influences tested for as required by 60601 standards.

## **Federal Communications Commission (FCC) Notice**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: reorient or relocate the receiving antenna; increase the separation between the equipment and receiver; connect the equipment into an outlet on a circuit different from that to which the receiver is connected; consult the dealer or an experienced radio or television technician for help. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

## **FCC RF Radiation Exposure and SAR Statements**

The RUBY 10 has been tested for body-worn Specific Absorption Rate (SAR) compliance. The RUBY 10 radio module has been evaluated under FCC Bulletin OET 65C (01-01) and found to be compliant with the requirements as set forth in CFR 47 Sections, 2.1093, and 15.247 (b) (4) addressing RF exposure from radio frequency devices. This model meets the applicable government requirements for exposure to radio frequency waves. The highest SAR level measured for this device was 0.15 W/kg.

Frequency range Wi-Fi: 2412~2472 MHz BT 2402~2480 MHz

Max. transmit power Wi-Fi: 18.0 dBm BT 5.0 dBm

## Industry Canada (IC) Notice

This Class B digital apparatus complies with Canadian ICES-003.

### Label Markings

WEEE icon

Fragile

This side up

Keep dry

CE mark

FCC mark

Warning/precaution

Power: ---5V.

Power rating

Manufacturer

Date of manufacture

Code 128

Barcode

Serial number

Authorized representative

Catalog number

Consult instructions

Medical device

Refer to instruction  
manual/booklet

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